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EN
PROPERTIES AND INDICATIONS

The disinfection and storage solution has an antibacterial function while maintaining an eye-friendly pH value. It is used for cleaning all rigid, gas-permeable contact lenses.

INSTRUCTIONS FOR USE

Before handling the contact lenses, wash your hands thoroughly. Remove lenses. Apply some drops of the disinfection and soaking solution to each lens surface. Gently rub the lens for a few seconds in the palm of the hand or between the fingers. Rinse the lenses with saline solution. Soak the lenses in the disinfection and storage solution for at least 4 hours or overnight. Before using the lenses, rinse them with saline solution.

PRECAUTIONS AND WARNINGS

Once opened, use within 90 days. Do not use the solution if the seal is broken, missing or tampered with. Store between 5°C and 30°C. Do not use the product after expiry date. Repeat the procedure if the lenses remain in the same solution for more than 7 days. Do not use tap water to rinse the lenses. Do not use the product if you are allergic or sensitive to any of the ingredients. Before taking any medication, consult your ophthalmologist. Keep out of reach of children. **DO NOT INGEST.**

CONTRAINDICATIONS

No contraindication reported. If you experience any irritation or any other adverse effects, remove lenses and consult your eye care specialist.

CONTAINS

Ethylene diamine tetra acetic acid 0.023%, PolyHexaMethyleneBiguanide 0.0002%, Polyvinylpyrrolidone, Polysorbate 80, Sodium chloride, Sodium Metabisulfite, Bibasic Sodium Phosphate, Monobasic Sodium Phosphate, Anhydrous Sodium Hydroxide, Purified water.

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